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The screenshot shows the 'Spirotrac® Database Studio' application. On the left is a dark blue sidebar with a menu. The 'BACK-UP DATABASE' option is highlighted with a red rectangle. The main window is titled 'Back-up maken van database' and contains the following fields and buttons:

- Server (Server\exemplaar):** A dropdown menu showing 'LT-AHUITING\SPIROTRAC'.
- Verificatie:** Radio buttons for 'Windows' (selected) and 'SQL'.
- Database:** A dropdown menu showing 'Spirotrac6'.
- Back-uplocatie:** A text field containing 'E:\Tijdelijk'.
- Buttons:** A 'BACK-UPLOCATIE' button next to the location field, and a 'BACK-UP' button at the bottom left of the main window area, which is also highlighted with a red rectangle.

At the bottom of the sidebar, the following information is displayed:

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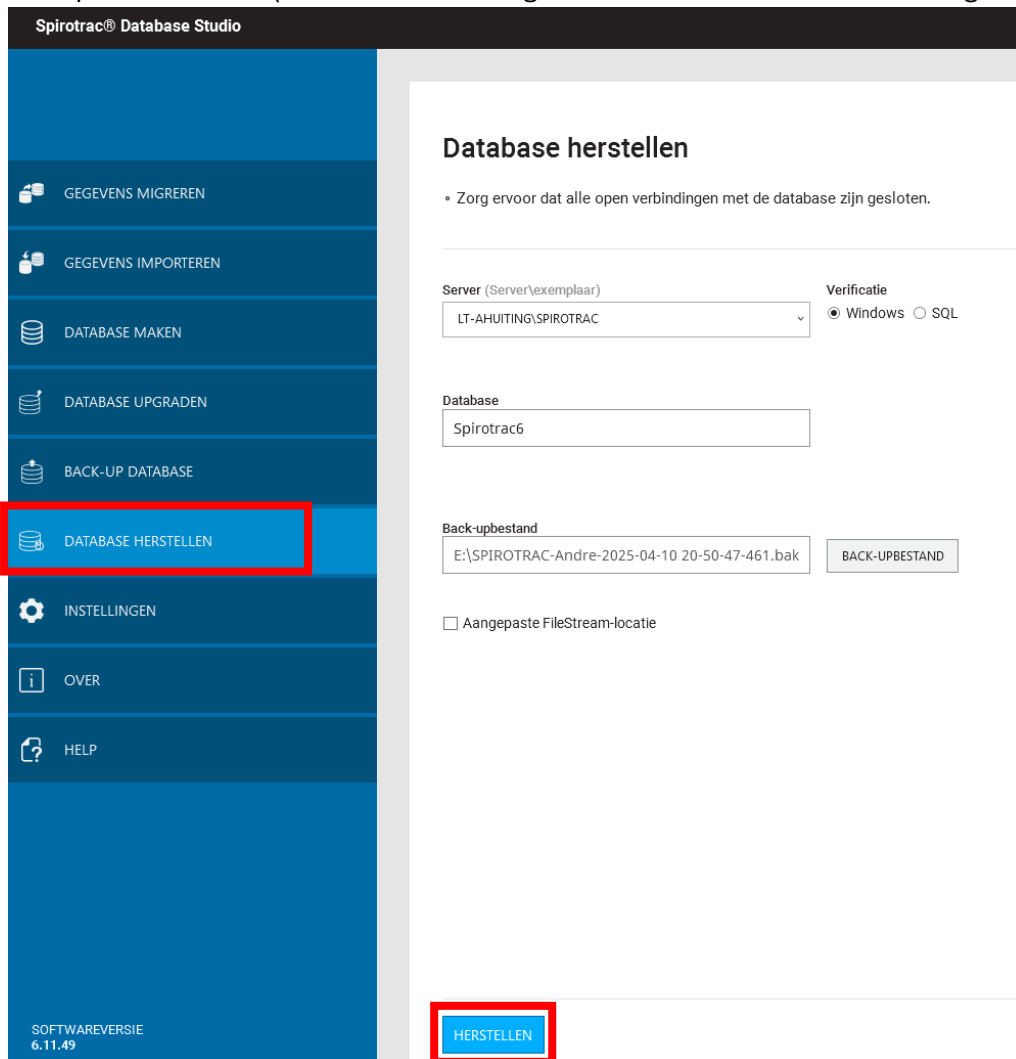
The screenshot shows the 'Spirotrac® Database Studio' application window. On the left is a dark blue sidebar with a menu. The 'DATABASE HERSTELLEN' option, which includes a database icon, is highlighted with a red rectangle. Below it are 'INSTELLINGEN', 'OVER', and 'HELP'. At the bottom of the sidebar, it says 'SOFTWAREVERSIE 6.11.49'. The main area is titled 'Database herstellen' and contains a warning: '• Zorg ervoor dat alle open verbindingen met de database zijn gesloten.' Below this are three sections: 'Server (Server\exemplaar)' with a dropdown menu showing 'LT-AHUITING\SPIROTRAC' and a small downward arrow; 'Verificatie' with radio buttons for 'Windows' (selected) and 'SQL'; 'Database' with a text box containing 'Spirotrac6'; 'Back-upbestand' with a text box showing 'E:\SPIROTRAC-Andre-2025-04-10 20-50-47-461.bak' and a 'BACK-UPBESTAND' button; and a checkbox for 'Aangepaste FileStream-locatie' which is unchecked. At the bottom right of the main area, a blue button labeled 'HERSTELLEN' is highlighted with a red rectangle.

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