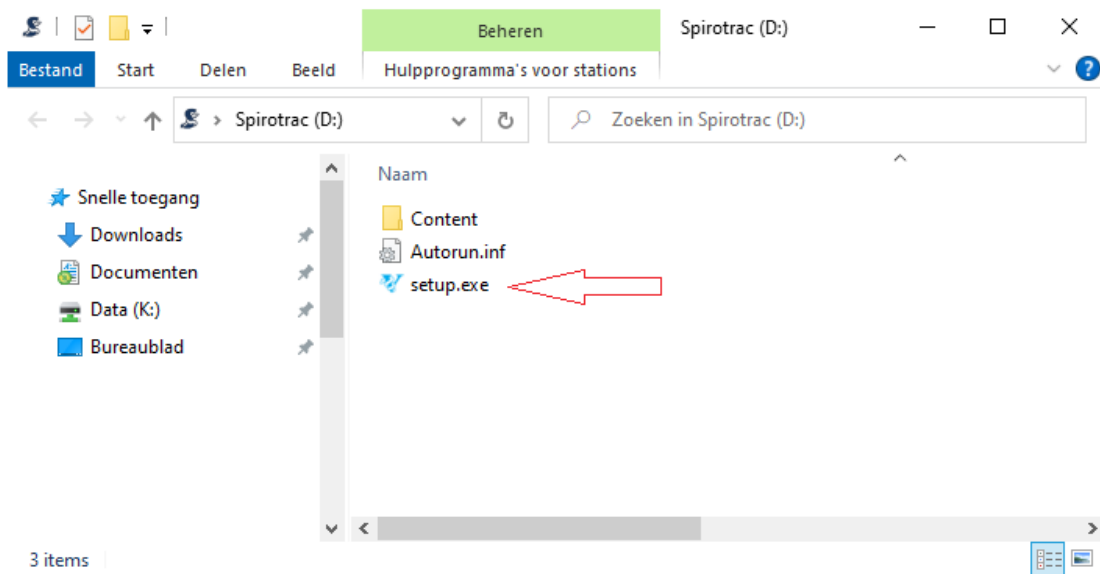
Installatie Spirotrac 6:

1

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
2

Dubbel klik op "setup"

3

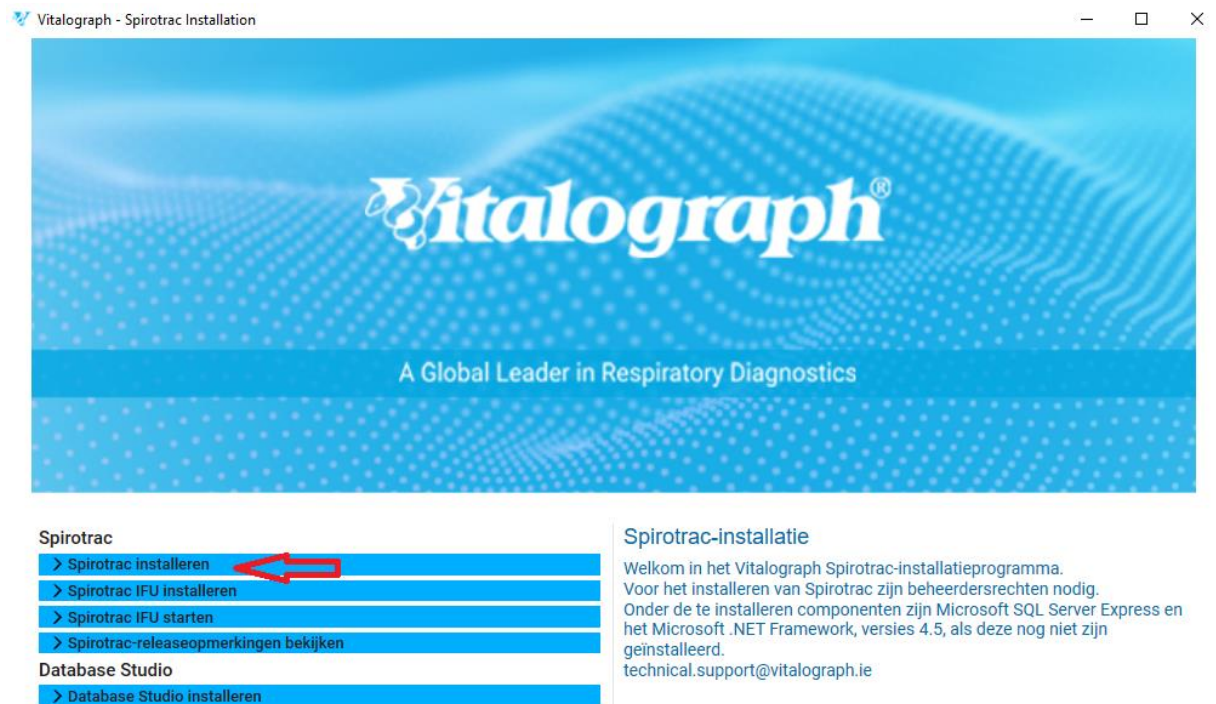Klik op "Spirotrac installeren"

4

Klik op "Volgende"

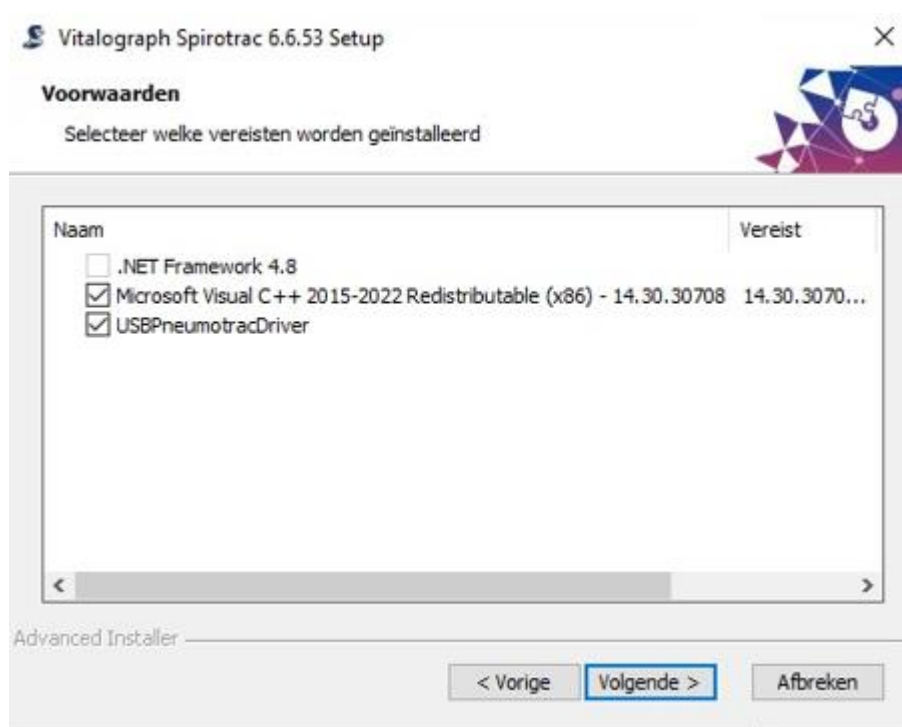
5

Klik op "Volgende"

6

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7

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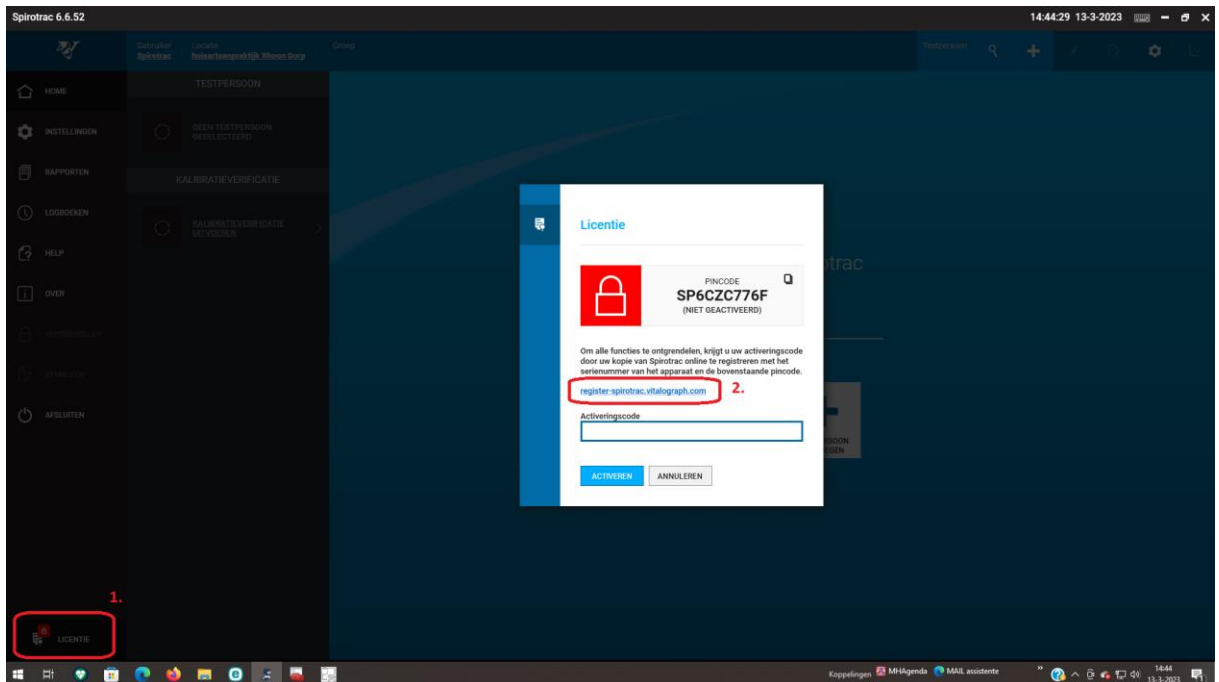
Spirotrac 6 Software Activation

This tool is used to activate Vitalograph Spirotrac 6 software to unlock all features of the product and download software updates.

To proceed and obtain an activation code, you will need to create a **Vitalograph account**. After creating your account you must register your **device serial number** first followed by your **Spirotrac PIN**.

LAUNCH SOFTWARE ACTIVATION TOOL

Find your device serial number

The serial number is 2 letters followed by 5 digits and is located on the product label on the device.

Find your Spirotrac PIN

1. Open Spirotrac 6.
2. Select the license icon at the bottom of the screen.
3. The PIN is displayed in the License window.

Once you have the activation code proceed as follows:

1. Open Spirotrac 6.
2. Select the license icon at the bottom of the screen.
3. Enter the activation code in the text box provided.
4. Click the "Activate" button.

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The screenshot shows a registration form titled "Inschrijven". At the top, there is a blue circular icon representing a user profile. Below the icon, there are two input fields: "E-mail" with the placeholder text "email@example.com" and "Wachtwoord". Below the password field, there is a link that says "Wachtwoord vergeten". At the bottom of the form, there are two buttons: "Nieuw account aanmaken" and "Inschrijven". A red arrow points to the "Nieuw account aanmaken" button, which is also highlighted with a red rounded rectangle.

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