One-Way Valve Mouthpiece with In-Check Dial G16

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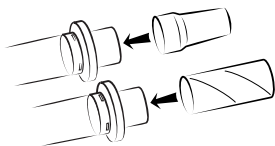
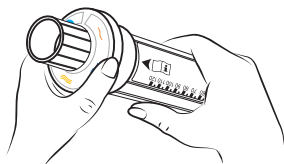
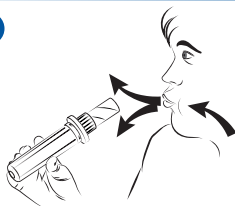
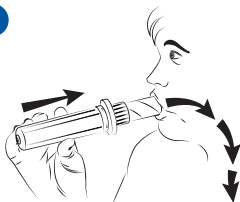
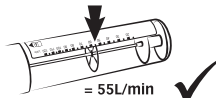
9

Intended Use

To enable healthcare professionals to coach patients to use their inhalers correctly. The device can simulate the resistance characteristics of many commercially available inhalers. Therefore, with the device set to the correct resistance, a patient can be trained to inhale at a flow rate known to be suitable for their personal DPI or pMDI.

Intended Purpose

A universal reusable, hand-held, oral device, intended to be used by healthcare professionals. The One-Way Valve Mouthpiece Kit with In-Check DIAL G16 is an inhalation airflow training system that can help educate and assess patients who use inhaler devices. It is designed to simulate the 'internal resistance' of common inhaler devices. Patients can be encouraged to modify their inspiratory technique (by inhaling with more or less effort), in order to achieve a flow rate consistent with their inhaler. It provides an assessment of inspiratory flow through five classes of device resistance. However, the resistance is not precise for all inhalers, e.g. the resistance of the Genuair is slightly lower than the resistance of the Turbohaler Mk III, but they are both medium resistance inhalers.

The In-Check DIAL G16 should only be used with a Clement Clarke approved adult single use or universal reusable mouthpiece. The One-Way Valve Mouthpiece with In-Check DIAL G16 is used in the assessment of technique for inhaled medication for patients with restrictive or obstructive respiratory conditions.

Contraindications

 Medical assessment is necessary and due care is to be exercised in cases of special needs patients.

The One-Way Valve Mouthpiece with In-Check DIAL G16 is not recommended for children under 10 years old.

The Universal Reusable Mouthpiece is not recommended for children under 5 years old.

The One-Way Valve Mouthpieces are not suitable for children over 10 years old who cannot comfortably form a tight seal around it with their lips.

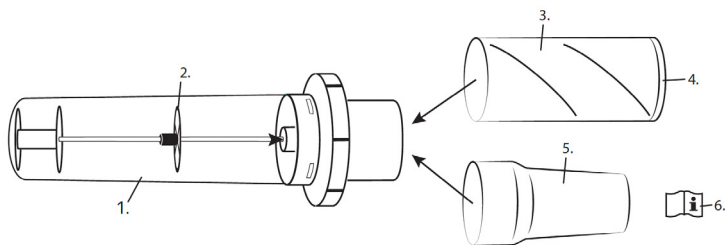
There are no other known contraindications for these devices.

Clinical Benefits

- A simple way for healthcare professionals to assess peak inspiratory flow rate (PIFR) values of patients and match them to resistance of inhalers.
- The indicator easily shows the reading against the calibrated scale.
- Lightweight, easy to use and portable allowing it to be used in a variety of healthcare settings without a compromise in performance.
- The transparent body enables a qualitative view of the inspiratory manoeuvre.
- This transparent material also makes it easier for the user to make a visual inspection for any damage or contamination before use.
- The range obtained from the device enables the patient to adjust their inhalation technique to best suit their own inhaler.
- Can be used on any patients over 5 years old with the universal reusable mouthpiece attached.
- Individually calibrated for accuracy.

Contents

1. 1 x In-Check DIAL G16
2. Indicator
3. 2 x Disposable One-Way Valve Inspiratory Mouthpieces
4. One-Way Valve
5. 1 x Universal Reusable Mouthpiece
6. 1 x Instructions for Use (IFU)

⚠ Warnings and cautions:

- **PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE (IFU) BEFORE USING THE DEVICE.**
- **BEFORE USE, THE IN-CHECK DIAL G16 MUST BE INSPECTED TO ENSURE THERE IS NO DAMAGE OR FOREIGN OBJECTS INSIDE WHICH COULD ACCIDENTALLY BE INHALED AND RESULT IN A RISK CHOKING.**
- If device or packaging is damaged on collection of the goods, contact Clement Clarke International Ltd. on +44 (0)1279 414969 or email resp@clement-clarke.com
- Any serious incident in relation to this device must be reported to the manufacturer and the Regulatory Authority of the country where the event occurred.
- If you have any questions about your One-Way Valve Mouthpiece or In-Check DIAL G16 and its use, talk to your physician or other licensed healthcare professional.
- To minimise cross contamination risks between patients, a Clement Clarke Inspiratory Disposable One-Way Valve Adult Mouthpiece is recommended.
- Universal Reusable Adult Mouthpieces can be used to channel the patient's airflow through the In-Check DIAL G16, if the whole device and mouthpiece are cleaned as described in this IFU between patients.
- The One-Way Valve Mouthpiece are intended for use by children over 10 year old. For younger patients a paediatric adaptor is recommended. Children between 3 and 5 years should only use after medical assessment.
- The information provided is for guidance only. It does not imply that any particular product will be clinically effective. There are other aspects than inspiratory flow that contribute towards clinical efficacy.
- Follow the Cleaning Instructions supplied.
- Do not autoclave the In-Check DIAL G16 as this will permanently damage the device.
- The instructions provided must be read and followed to obtain the correct measure of airflow.
- The In-Check DIAL G16 does not require further calibration, the device is supplied already calibrated and is suitable for 2 years following its first use. After 2 years, the device must be replaced to guarantee correct measurement.

- If you are unable to obtain a reading you should contact your licensed healthcare professional immediately.
- To minimise cross contamination, ensure the appropriate Mouthpiece has been securely attached:
 - Clement Clarke International single-use one-way valve Inspiratory Mouthpiece.
 - Universal Reusable Mouthpiece can also be used if the In-Check DIAL G16 and mouthpiece are both decontaminated between patients, see the Cleaning Instructions supplied.
- If required, a nose clip can be used to help block the nostrils when using the In-Check DIAL G16.
- **Keep original packaging and store cleaned In-Check DIAL G16 in it in a clean and dry location away from the risk of accidental damage or exposure to any hazardous substances and to minimize the risk of contamination by foreign objects resulting in choking.**
- **DO NOT LEAVE CHILDREN UNATTENDED WITH THE IN-CHECK DIAL G16**
– Risk of breakage / Choking.

IF YOU DO NOT UNDERSTAND THE IFU, PLEASE CONSULT CLEMENT CLARKE INTERNATIONAL LTD., CALL: +44 (0)1279 414969, OR WRITE TO resp@clement-clarke.com

Performance Characteristics

The In-Check DIAL G16 has been designed applying the applicable parts of EN ISO 23747:2015 Anaesthetic and Respiratory Equipment.

- Range 15 – 120 L/min
- Accuracy $\pm 10\%$ or 10 L/min (whichever is higher)
 - Repeatability of ± 5 L/min

The In-Check DIAL G16 is individually calibrated to ensure accuracy.

 The In-Check DIAL G16 is not a serviceable device.

How the In-Check DIAL G16 Works

The In-Check DIAL G16 is a low-range inspiratory flow device that has a selectable resistance from high to low, shown by the coloured 'flow' icons on the back cover of this booklet.

During inhalation the indicator inside the In-Check DIAL G16 slides along the calibrated scale and stops at the patients PIFR value.

This indication helps the physician or other licenced healthcare professional to determine the most appropriate inhaler device or assess whether the patient's device is the most appropriate for them to use.

 **Changes in your airflow may require special treatment according to a Treatment Plan given to you by your physician or other licensed health care professional.**

How to set the In-Check DIAL G16

To set the In-Check DIAL G16 hold the device vertically with the mouthpiece end uppermost so the rounded end of the meter can be tapped firmly from underneath with the palm of the hand. The magnetic weight will be dislodged and will return the indicator to the start position with a notable knock noise. When this has happened, invert the meter to return the magnetic weight to its resting position.

Warnings - Preparation for use

- Stand or sit upright, choose what position is most comfortable and always do it that way.
- Do not bend your neck.
- Do not put your tongue into the mouthpiece.
- Do not cough or spit in the mouthpiece.

How to use - see diagrams

1. Attach In-Check DIAL G16 to a clean mouthpiece, enclosed in the carton.
(Only CCI mouthpieces are recommended for the In-check DIAL G16).
2. Align the dial selector with the desired coloured icon – an audible “click” should be heard.
3. Reset the In-Check DIAL G16. See ‘How to set the In-Check DIAL G16’ section.
4. Ask the patient to exhale fully.
5. Ask the patient to seal their lips around the mouthpiece. According to the inhaler chosen, instruct the patient to inhale in the manner recommended by the inhaler manufacturer.

To operate an inhaler device correctly, the patient should be able to achieve a flow rate within the clinically effective range.

- e.g. Accuhaler: Clinically Effective Flow Rate 30-90 L/Min.
6. Record the inspiratory flow from the position of the indicator against the gauge. Reset, and repeat two more times, ensuring correct technique each time.
 7. Compare results achieved with target flows for the selected device.
 8. If after repeated training the patient is not able to achieve a consistent flow range, then the healthcare professional may wish to assess the patient’s ability to use an alternative type of inhaler.
 9. Dispose of any single-use mouthpieces in general waste.

Troubleshooting

Table 1: Troubleshooting

Fault state	Possible cause	Actions to take
Readings are out of the recommended range / incorrect readings	The instructions were not followed correctly	Read these instructions for use
	The In-Check DIAL G16 is broken	Stop using the device
	The correct accessories have not been used	Ensure the correct sized Mouthpiece has been used (see Warnings and cautions section)
	The indicator has not been reset properly	Set the In-Check DIAL G16 by following the steps described in 'How to Set the In-Check DIAL G16' section
The indicator will not move when inhaling through the device	The In-Check DIAL G16 is broken	Stop using the device
	The instructions were not followed correctly	Set the In-Check DIAL G16 by following the steps described in 'How to Set In-Check DIAL G16' section
The setting weight does not move	The device may not be set to the starting position	Set the In-Check DIAL G16 by following the steps described in 'How to Set In-Check DIAL G16' section
	The In-Check DIAL G16 is broken	Stop using the device
Cannot get device to work / don't know how to use the device	The In-Check DIAL G16 may be broken	Re-read these instructions for use
	Unable to read or otherwise understand the instructions	

If you have any questions about whether your In-Check DIAL G16 is working correctly, call Clement Clarke International Ltd on +44 (0)1279 414969 or email resp@clement-clarke.com

Cleaning Instructions

It is recommended to clean these devices after each use and store in their original packaging to minimise the risk of contamination by foreign objects resulting in a risk of choking.

 The one-way valve inspiratory mouthpieces are for single patient – multiple use and should be disposed of after each patient.

Do not:

- clean the In-Check DIAL G16 with a brush, cloth or any other implement as this will permanently damage the device
- autoclave the In-Check DIAL G16 as this will permanently damage the device

Do:

- keep the In-Check DIAL G16 and universal mouthpiece clean when not in use to prevent bacterial build up
- clean the In-Check DIAL G16 and Accessories daily when in use

Disinfection Frequency Recommendations

The following recommendations for disinfection frequencies are presented as a guide only. In practice, the person responsible for the clinical wellbeing of the patient should consider the specific circumstances of the next patient and the risk of infection posed by cross-contamination.

-  Dispose of any Single Use / Single Patient – Multiple Use Mouthpieces in the designated waste bin.

The In-Check DIAL G16 and reusable accessories can be cleaned by immersing the devices in a solution of Lancerzyme and clean water at or below 30°C. They can then be disinfected using Tristel 1 Day solution. The dilution rates and exposure times shown on the labelling of the detergent and disinfectant must be followed. When immersed, agitate the devices to ensure thorough cleaning.

Rinse thoroughly in clean water at or below 30°C, and gently shake to remove any excess fluid.

-  It is important to rinse thoroughly to prevent salt spots appearing on the inside of the devices and prevent damage to the spindle.

Hold the device at the end furthest away from the mouthpiece then gently shake to remove any excess water. Keep the device vertical to allow the water to drain from the open end. Stand the device upright on its open end and allow to air dry thoroughly for between 30 to 60 minutes before reuse or storage. Once dry, store in its original packaging in a clean, dry place away from any hazardous substances.

If required, the In-Check DIAL G16 can be sterilized using the Low Temperature hydrogen peroxide plasma Sterilization System (H202) after the washing process has been carried out.

Warranty Information

For warranty consideration, devices must be returned freight prepaid, with proof of purchase date, to Clement Clarke International Ltd.

Devices damaged accidentally or subjected to misuse are excluded from this warranty. The manufacturer reserves the right to replace devices at its discretion under the terms of this warranty.

The foregoing is the sole warranty provided by Clement Clarke International Ltd. in connection with this product. Clement Clarke International Ltd. hereby disclaims any other warranties, expressed or implied, including warranties on merchantability and fitness for particular purpose. Implied warranties and other terms that may be imposed by law, if any, are limited in duration to the period of the above express warranty. Clement Clarke International Ltd. shall not be liable for loss of use or any other special, incidental, consequential or indirect costs, expenses or damages.

Disposal

All Mouthpieces and the In-Check DIAL G16 can be disposed of in general waste.

-  The packaging and Instruction leaflet are fully recyclable.

Part Codes

Disposable one-way valve mouthpieces	(Inspiratory, 5 x 20)	3122061
Disposable one-way valve mouthpieces	(Inspiratory, bulk, 200)	3122063
Universal Plastic Mouthpieces	(15)	3103095
Adult to Paediatric Mouthpiece Adaptor	(20)	3122120
One-Way Valve Mouthpiece Kit with In-Check DIAL G16	(1)	3109300

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Clement Clarke International Ltd.

Cynon Valley Business Park • Mountain Ash • CF45 4ER • UK

+44 (0)1443 474647 • resp@clement-clarke.com • www.clement-clarke.com

Flexicare Deutschland GmbH

Ohlenhohnstrasse 34 • 53819 Neunkirchen-Seelscheid • Germany

Flexicare Medical Switzerland GmbH

c/o BK-Services AG Suurstoffi 22 • 6343 Rotkreuz Switzerland

Clement Clarke International Ltd is part of the Flexicare Group.

Made in Sri Lanka | 3109305 Iss 12 | May 2023

Icon	Product	Icon	Product	Device Resistance
	Handihaler®		Forspiro®	
	Easyhaler®		Elipta®	
	NEXThaler®		TurboSpin®	
	Twisthaler®		Diskhaler®(4)	
	Turbohaler® P		Accuhaler®	
	Turbohaler® S		Breezhaler®	
	Clickhaler™		k-haler®	
	Spiromax®		Respimat®	
	GenuAir®			

Please Note:

All trademarks and product names are the property of their respective owners, see IFU booklet for details.

Assessing inspiratory flow rate for clinical efficacy:

Select appropriate resistance setting, inhale through meter, assess achieved flow rate.

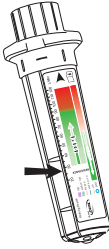
For DPIs values between 30-90 L/min are generally associated with clinical efficacy.

For pMDIs values between 20-60 L/min are preferred.

For Handihaler 20 L/min is associated with clinical efficacy.

Bibliography:

1. Pedersen S, Dubus JC, Crompton G. 5) Inhaler selection in children with asthma. *Prim Care Respir J* 2010;**19** (3):209-16
2. Lavorini F, Levy ML, Corrigan C, Crompton G. 6) Training tools for inhalation devices. *Prim Care Respir J* 2010;**19** (4):335-41
3. Capstick, T. G., & Clifton, I. J. (2014). Inhaler technique and training in people with chronic obstructive pulmonary disease and asthma. *Expert review of respiratory medicine*.

4. Sanders, M. J. (2017). Guiding Inspiratory Flow: Development of the In-Check DIAL G16, a Tool for Improving Inhaler Technique. *Pulmonary medicine*, 2017.

5. Mahler, D. A. (2017). Peak inspiratory flow rate as a criterion for dry powder inhaler use in chronic obstructive pulmonary disease. *Annals of the American Thoracic Society*, 14(7), 1103-1107.

6. Laube, B. L., Janssens, H.M., de Jongh, F.H., Devadason, S.G., Dharm, R., Diot, P., ...& Chrystyn, H. (2011). What the pulmonary specialist should know about the new inhalation therapies. *European Respiratory Journal* **37** (6) 1308-1417.

Inhaler Resistance Range

International

- Handihaler®
- Easyhaler®
- NEXThaler®
- Twisthaler®
- Turbuhaler®
- Turbuhaler®, Flexhaler®
- Clickhaler™
- RespiClick®, Spiromax®
- Novolizer®, Genuair®, Pressair®
- Forspiro®
- Ellipta®
- TurboSpin®
- Diskhaler®
- Diskus®
- Breezhaler®, Aerolizer®, Neohaler®
- k-haler®
- RespiMat®