**ENGLISH (en) – Product Description – Hans Rudolph 7450 Series V2 Mask with Mask Adapter for Vyaire Medical Flow Meters**

**Intended Use - Environment for Use:**

The 7450 series face mask is a reusable, multi-patient, multi-use mask intended for use with two-way non-rebreathing valves or other mask adapters for connection with flow measuring instruments used in lung function testing procedures with spontaneous breathing adults (>30 kg) in hospitals and other clinical settings by individuals that have at least minimal training or instructions on the use of the mask as well as the devices to which the mask is intended to connect. Applications are for pulmonary function testing, exercise testing and respiratory research. The masks, valves, adapters and headgear are supplied clean, non-sterile.

**Cautions**  
Federal law restricts this device to sale by or on the order of a physician

**Warnings**

- Shipped clean, non-sterile. If disinfected or sterile use is required, follow the procedures in this document prior to use of the mask.
- To avoid skin irritation, never use any cleaning solutions or chemicals on this mask or headgear other than a mild detergent and water. Discontinue use if irritation or allergic reaction develops due to the mask.
- Patients with facial hair may experience leakage.
- Do not use these masks, valves and or adapters in a life support system such as a ventilator circuit.

**Materials**

- Face Piece is silicone rubber with polycarbonate plastic structural braces for headgear strap clip attachments
- Mask Adapter is silicone rubber
- Headgear is a laminate of polyurethane foam with nylon fabrics
- Headgear Tri-Glide and Strap Clips are polypropylene plastic

**Recommend Headgear Sizes (L, M & S)**

**Mask Size (Headgear Size):** L(L) M(M) S(S) XS(S) P(S)

**Deadspace ml (Total Mask & Mask Adapter)**

**Mask Size (Deadspace ml):** L(160) M(142) S(116) XS(105) P(95)

**Service Life**

Mask, mask adapter and headgear when maintained and reprocessed per these instructions are expected to stay in service for minimum of 25 disinfection or steam sterilization cycles and the headgear 25 cleaning cycles. Visual inspection of the components during reprocessing will identify any defects. If defects are identified dispose of and do not use.

**Operational Conditions for Use**

Temperature range: 5-40° C  
Humidity range: 0-95% RH

**Mask Components (fig 1)**

- Mask face piece has plastic braces with slot openings for attachment of the headgear strap quick-release clips
  - Headgear has adjustable tri-glide on top of head and quick-release connection clips with strap locking adjustments
  - Mask Adapter is a flexible silicone rubber designed to attach to all of the Vyaire Medical flow meters
- Application & Fitting (fig 2)**
- Determine patient mask size with optional Mask Sizing Gauge (fig 3)
  - Place mask over patient's nose and mouth (1). Chin should fit into mask chin cup portion with top of mask sealing area on the bridge of the nose.
  - Slide headgear over patient's head (2) with one of the lower quick release strap clips disconnected. Reconnect headgear strap clip after headgear is mounted (3). Bottom straps should be positioned below ears and top straps above ears and below eyes.
  - Adjust headgear strap tension by pulling slightly away from the face on strap end hook sections until tension is evenly distributed at all four straps with mask snug but comfortable (4 & 5). The strap clips lock and hold the position of the strap. To loosen the straps pull up on the back of the clip. Attach hook ends to the headgear fabric to keep the ends attached and out of the way. Adjust straps at crown of head by pulling each in opposite direction until reaching the required tension then connect the Velcro™ (6).
  - Push the flow meter patient side into the port opening in the flexible mask adapter

**Disassembly for Cleaning**

- Separate headgear from mask
- Separate mask adapter from mask
- Do not remove the upper and lower plastic braces (headgear attachment) from the mask face piece

FIGURE 1		ESPAÑOL (es)	DEUTSCH (de)	FRANÇAIS (fr)
A	Adaptador de la mascarilla	Maskenadapter	Adaptateur du masque	
B	Pieza de la cara	Gesichtsteil	Pièce faciale	
C	Abrazaderas estructurales (2)	Verstärkungen (2)	Attaches plastic (2)	
D	Ranuras de ajuste de los broches de las correas del arnés para la cabeza (4)	Schlitzze zum anbringen der kopfhauben-haltebänder -clips (4)	Encoches d'ancrage des courroies à clips. (4)	
E	Arnés para la cabeza	Kopfhaube	Harnais	
F	Broches para bloqueo de las correas de liberación rápida (4)	Quick Release Clips der Haltebänder (4)	Clips à ouverture rapide des courroies (4)	
G	Hebilla triple	Leiterschleife	Tri-Glide	

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EC REP M.Devices Group / EC Rep Ltd  
Healthcare Education Centre  
Portland Street  
Southport PR8 1HU, UK  
P-44 1704 544 944  
Email: info@ecrep.com

PRINTED IN U.S.A.  
691341 2018-12-03 Rev. B

HANS RUDOLPH, INC.  
8325 Cole Parkway  
Shawnee, Kansas 66227 USA  
Tel: (913) 422-7788  
Toll Free: (800) 456-6695  
Fax: (913) 422-3337  
Web Site: www.rudolphkc.com  
Email: hri@rudolphkc.com

CE 0086 © 2018 Hans Rudolph, inc.

US Pat No's D573,709 & 8,136,523 and  
EC Req'd Design No. 000780556-001

**Cleaning Mask face piece and Mask Adapter**

- Soak components for 5 minutes in warm water with mild detergent (neutral pH)
- Hand wash components with sponge or soft brush; then rinse with warm tap water
- Place components in warm water bath and agitate for two minutes
- Rinse in clean tap water for at least 1 minute
- Allow components to completely air dry or dry with clean, lint free cloth

**Cleaning Headgear**

- Submerge headgear in warm soapy tap water; gently rub all areas
- Rinse in clean tap water for 1 minute or until all soap is removed; air dry
- Precaution: Do not use bleach, chlorine or alcohol based solutions to clean mask or headgear. These solutions can damage components. Direct sunlight exposure on mask and headgear components can cause deterioration and reduce product life.

**High Level Disinfection (Mask & Mask Adapter only)**

- Thoroughly clean and dry the mask and mask adapter prior to disinfection or sterilization
- For high level disinfection use liquid glutaraldehydes (Cidex™) or hot water pasteurization.
- Do not use any alcohol or chlorine based solutions.

**Liquid Chemical Disinfection**

- Soak the device components in the liquid chemical solution per the manufacturer's instructions.
- Remove the components and soak in 1500 ml of water (preferably sterile) for a minimum of 1 minute.
- Repeat step 2 and then dry the device with a clean lint free cloth (preferably sterile)

**Hot Water Pasteurization**

- Immerse all device components surfaces in hot water temperatures 71.1° C – 76.6° C for 30 minutes.
- Dry the components with a clean lint free cloth (preferably sterile)

**Steam Sterilization (Silicone Rubber Face Mask & Silicone Rubber Mask Adapter less Headgear)**

- Thoroughly clean and dry the mask and mask adapter prior to disinfection or sterilization.
- Mask and mask adapter should be assembled

**Steam Sterilization Pre-Vacuum Cycle**

- Temperature: 132° C
- Sterilization Time: 4 minutes
- Dry Time: 10 minutes

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