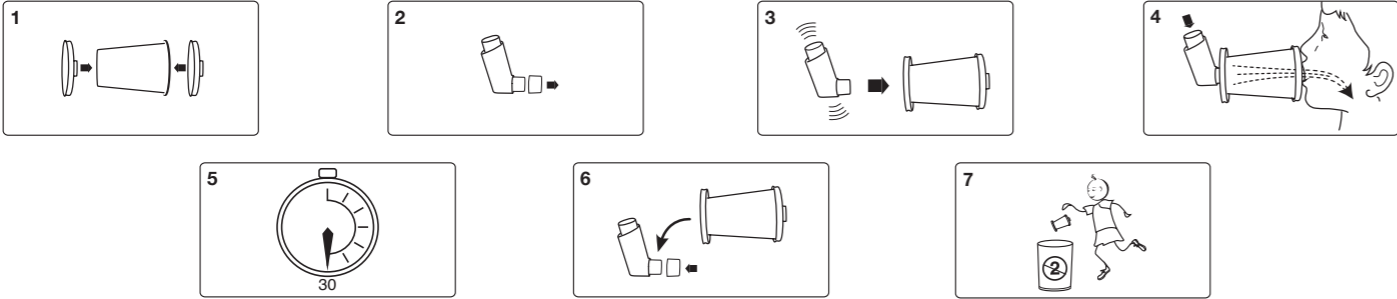
## DispozABLE™ Spacer

#### sv

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#### Symbols

	Consult accompanying documents
	Manufacturer
	Single use <span> </span> : use only once
	This product complies with the essential requirements of the medical devices directive (93/42/EEC).

 Clement Clarke International Ltd.,  
Edinburgh Way, Harlow, Essex.  
CM20 2TT, GB.  
Tel: +44 (0)1279 414969  
Fax: +44 (0)1279 456300  
Website: www.clement-clarke.com

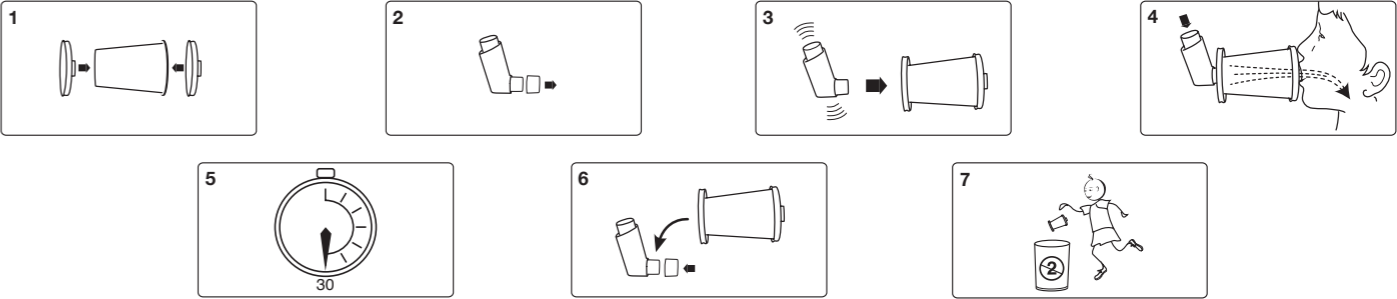
Part no: 3609510 issue 2 Jan 2021

**EC REP**

**HAAG-STREIT SURGICAL GmbH & Co. KG.**  
Rosengarten 10, 22880 Wedel, Germany.  
T: +49 4103 709 01

**HS CLEMENT CLARKE INTERNATIONAL**

## DispozABLE™ Spacer - Instructions for Use

#### en

- Assemble the DispozABLE™ Spacer. Ensure no objects have been left inside the DispozABLE™ Spacer prior to use.
- Remove cap from MDI.
- Shake MDI/Prepare the MDI as per manufacturer's instructions. Insert MDI into MDI adapter at base of DispozABLE Spacer.
- Breathe out all the way, place DispozABLE Spacer mouthpiece between lips. Actuate MDI (one puff). Immediately inhale slowly through the mouth. Hold breath for 5-10 seconds. Breathe normally.
- Wait 30 seconds/ Follow manufacturer's instructions. Repeat steps 2 to 4. Introduce one puff at a time into Spacer.
- After use, remove MDI and replace its cap.
- It is recommended DispozABLE Spacer is disposed of after each use. Can be disposed of with normal household waste.

**For use with Metered Dose Inhalers.** The DispozABLE Spacer has been designed for use in schools, in an emergency and forreversibility testing. Designed to improve medication deposition improving the delivery of medication from your metered dose inhaler (MDI) deep into the lungs, which in turn makes the medication more effective. This Spacer should be disposed of after each use.

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